


ProstaSIM

INSTRUCTIONS

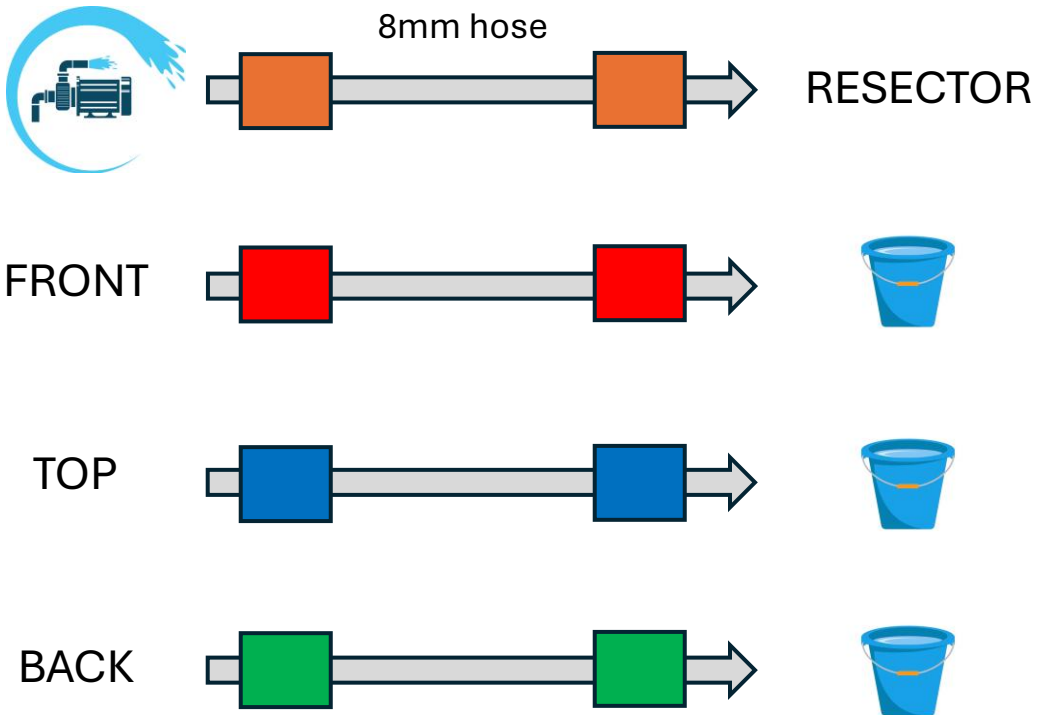

- Place each collared hose according to the diagram.
- Place the water pump provided inside an appropriate container.
- To properly fill the bladder at the beginning, you may need to pinch the back hose (green) momentarily until water flows out of the bladder from the upper hose (blue).
- If the flow of water or saline solution is blocked in any of the hoses, replace it with a standard 8mm silicone hose.
- To replace the prostate phantom, unscrew the urethra section, remove the used phantom and place the new one inside the opening. Use the arrow as reference.
- Clean and dry the simulator box, bladder and tray after each use.
- We recommend using the provided case to store your ProstaSIM.

